



Make you see with joy!



Acumed Source, Inc.  
Tel : 626-279-5158  
Fax: 626-401-1889

## Vision Care Eye Massager

### User manual

Please read the entire manual carefully before using your eye care massager and follow the instructions for proper usage.

This device should not be used by: children younger than 3 and half years of age, people with eye disease, eye surgery, or severe myopia, and those who suffer from motion sickness.

#### The package contains:

- Main unit
- Non-prescription clear lenses
- Adapter
- USB cord
- User manual
- Pupillary distance adjustment buckle

#### Product Components:

- Frame
- Adjustment buckle
- USB jack
- USB connector cord
- Adapter
- Silicone band
- Silicone massage pad

#### Technical Specifications

- Power supply: 5V/1.0A-1.2A
- Power: 10W
- Massage session length: 10 minutes
- Ideal environment: Temperature 32F – 104F; Relative humidity: : ≤80%
- Massage action: continuous
- non-AP/APG style



Colours



## Directions how to use

- 1.Plug one end of the USB cord into the USB jack on the top right side of the eye massager.
- 2.Plug the other end of the USB cord into the included adaptor or to the USB outlet of a mobile charger, car charger, or computer (Make sure to only use with power outlets at 5V/1.0A-1.2A to ensure safe use of the device).
- 3.During the first 15 seconds of operation, the device will adjust itself to massage pressure points. Afterwards, it will automatically start massaging and heating. (Heat temperature: 107°F +/- 5°F);
- 4.Adjust the Pupillary Distance Adjustment Buckle as necessary to ensure proper fit. Wiggle the buckle gently and adjust to your size. Do not use excessive force.
- 5.The massager stops automatically after a 10-minute session. If you desire another session, unplug the cord, and re-plug it in again. A new 10-minute session will start.
- 6.It is normal to feel the pressure, tightness, itchiness or even mild pain while using the massager. A few sessions of usage will correct this. You may also massage around your eyes gently, or step outside to look at flowers or green plants to recover. Stop use if discomfort continues.

This device has a functional model and a comfortable model.

## Transportation and Storage

- Transportation: Secure package to prevent severe shaking and moisture
- Storage temperature: 32°F to 104°F
- Relative humidity: : ≤80%

## Precautions

- Do not use the massager in excess of 30 minutes at a time to prevent discomfort.
- Do not fold the pupillary distance adjustment buckle as it may break as a result.
- If one side does not vibrate, tap slightly on that side to activate vibration.
- If the massage does not function continuously, unplug and re-plug the USB cord.
- When necessary, unplug the USB adaptor. Limit the unplugging and re-plugging of the small USB jack.
- It is normal to have mild red mark on skin around your eyes after using the massager. It is from the wearing belt and will disappear shortly after.
- In case of any question for the device, please contact your seller/distributor.

**Warnings: Do not try to open the main unit that should only be opened by an authorized personnel of our company.**